kitett®

FISIO® PRO

FR – Notice d'utilisation
EN – Instructions for use
DE – Gebrauchsanweisung
NL – Gebruiksaanwijzing
IT – Istruzioni per l'uso
AR – تعليمات الاستخدام

K.NU.MU.157.0823 – Version 16, August 2023

CE 0459 (2003)

DTF MEDICAL

LA DIFFUSION TECHNIQUE FRANCAISE
19 rue de la Presse, CS 60132, 42003 SAINT-ETIENNE Cedex 1 France
Phone number: +33.(0)4.77.74.51.11
Email: dtf@dtf.fr – www.dtf.fr

FR – Notice d'utilisation	3
Tire-lait électrique associé au KIT EXPRESSION KOLOR®	
EN – Instructions for use.....	14
Electric breast pump associated with KOLOR® KIT EXPRESSION	
DE – Gebrauchsanweisung	25
Elektrische Milchpumpe in Verbindung mit KOLOR® KIT EXPRESSION	
NL – Gebruiksaanwijzing	37
Elektrisch borstkolfapparaat geassocieerd met KOLOR® KIT EXPRESSION	
IT – Istruzioni per l'uso.....	49
Tiralatte elettrico in abbinamento con il KIT EXPRESSION KOLOR®	
AR – تعليمات الاستخدام	70
مضخة الثدي الكهربائية تتعلق بجهاز KOLOR® KIT EXPRESSION	

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Dispositif Médical

1. Intended use	15
a) Indications	15
b) Contraindications and adverse events	15
c) Users	15
2. Product description	15
a) Detailed view and description	15
b) Selecting the breast shield size	16
3. Cleaning, disinfection and maintenance	16
a) Breast pump cleaning and maintenance	16
b) Accessories cleaning at home	16
c) Disinfection at home	17
d) Cleaning and sterilization by healthcare establishments	17
4. Use	18
a) Breast pump functioning	18
b) Accessories preparation	18
c) Using the breast pump	18
d) After each use	19
5. Breastfeeding	19
a) Storing breast milk	19
b) Baby's breastfeeding physiology	19
6. Safety precautions	19
7. Problem solving	20
8. List of accessories and spare parts available	21
9. Warranty / Recycling	22
10. Technical specifications	22
11. Electromagnetic compatibility (EMC)	23
12. Pictograms	24

1. Intended use

a) Indications

Breast pumps are designed to be used by lactating women in order to extract their maternal milk. They help them to relieve their mastitis or breast engorgement and facilitate their healing. They also help mothers to relieve cracked or painful nipples or when breastfeeding is insufficient or impossible (inverted or painful nipples, etc.). Furthermore, they give the possibility to mothers to breastfeed their baby if they cannot be directly breastfed (preterm birth or neonatal pathologies (cleft lip and/or palate), difficulty of the baby to suck, etc.). Finally, they allow to initiate or maintain lactation, especially in case of a medical treatment incompatible with breastfeeding.

b) Contraindications and adverse events

There are no known contraindications or adverse events for the use of the FISIO® PRO electric breast pump.

Adverse event: a risk of allergy with the materials (thermoplastic elastomer) in contact with the body parts can appear.

c) Users

The breast pump is intended for use by any breastfeeding woman, whether an adult or a teenager (accompanied by an adult). The minimum requirement is to have read these instructions and to have used electrical appliances before.

2. Product description

The FISIO® PRO breast pump is an electric pump. It can be used in case of temporary separation or any other reason making breastfeeding impossible or not sufficient. Easy to use, it is ideal for active mothers.


The FISIO® PRO pump can be used in single or double pumping and should be plugged into the power supply. It combines a suction power adjustment with a set-up rhythm. The rhythm and the suction power can be **independently** adjusted. It tends to reproduce the baby's sucking rhythm: the mother experiences the stimulation and expression phases.

All KOLOR® breast shield sizes connect to the breast pump. The mother can thus have a suitable sized breast shield for her breast morphology to provide optimal comfort for an effective, long-lasting breastfeeding experience.

The breast shield is a one-piece which favours an optimized hygiene, and makes it possible not to touch the thread nor the internal parts of the feeding collection container and the breast shield.

a) Detailed view and description

The breast pump design is presented on the first page. The user interface is presented as below:

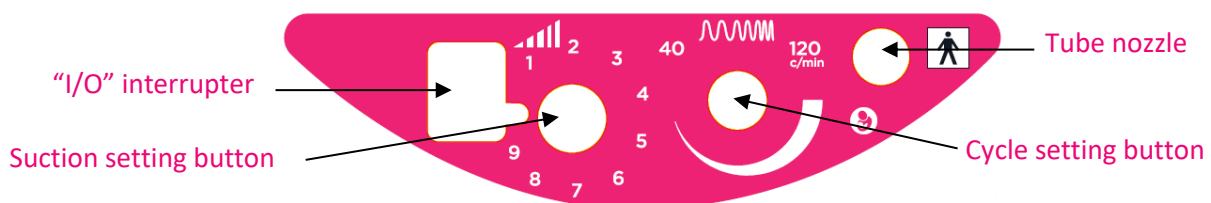Single KIT EXPRESSION: contains 1 KOLOR® breast shield, 1 collection container, 1 cap, 1 interface bottle and 1 PVC tube.

Double KIT EXPRESSION: contains 2 KOLOR® breast shields, 2 collection containers, 1 interface bottle, 2 PVC tubes and 2 caps.

Every KIT EXPRESSION and spare parts available and compatible with the FISIO® PRO breast pump are presented in Part 8.


NOTE: KOLOR® breast shields, collection container, cap and interface bottle are in polypropylene and thermoplastic elastomer. KOLOR® breast shields are guaranteed BPA and Phthalate free.


b) Selecting the breast shield size

The FISIO® PRO breast pump is used with the KIT EXPRESSION. The KOLOR® breast shield is available in 4 inner diameters and 2 «comfort» sizes that adapt perfectly to the breast morphology and guarantee a high level of comfort.

4 inner diameters: 4 colour codes

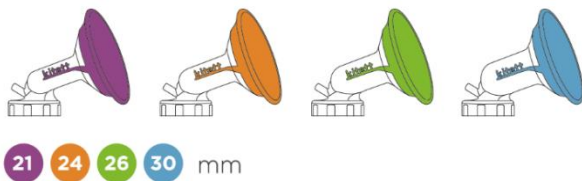
2 comfort sizes: S = 76 mm and L = 90 mm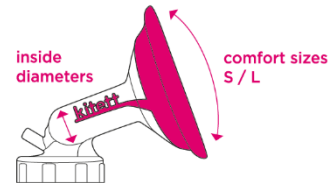

KITETT® helps you choose the breast shield using:

- the Kolorama size ruler

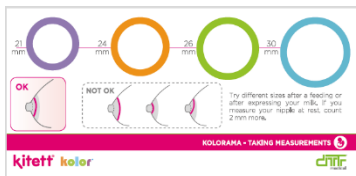
- the KOLORYOU® application available for free download

Measurements:

Measurement should be taken after breastfeeding or expressing your milk. Alternatively, if your nipple is at rest, add an extra 2mm around it to select the correct inner diameter size.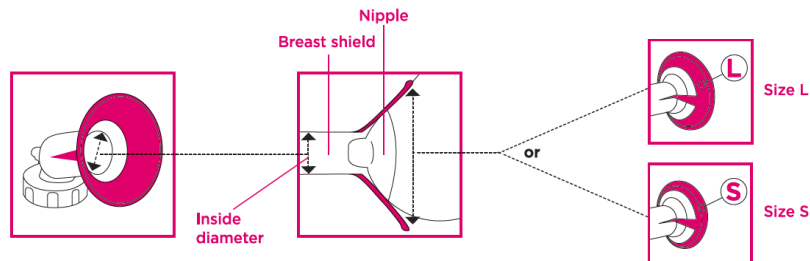

The nipple should not feel too constricted neither too free. You should see space around your nipple in order to let it move easily in the breast shield during pumping. If the breast shield is too tight, it could cause pain, lesions or compressed ducts. If the breast shield is too wide, less milk will be expressed. A correct comfort size (S -Small- or L -Large- comfort shape) will improve breast pump efficiency (optimal airtightness).

3. Cleaning, disinfection and maintenance

a) Breast pump cleaning and maintenance

Disinfection is done with a clean and damp cloth that is impregnated with a surface agent disinfectant for medical devices. Follow the instructions provided by the supplier of the surface disinfectant agent. Do not use abrasive cleaning agents. Unplug the device before cleaning.

b) Accessories cleaning at home

Accessories concerned:

- KOLOR® breast shield
- Collection container
- Cap

Before using for the first time and after each use:

- ① Disassemble each element.

- ② Rinse every part in contact with the milk in cold water (about 20°C).
- ③ Then clean with hot and soapy water.
- ④ Rinse every part.
- ⑤ Let them dry in an open air on a paper towel or a clean linen.

NOTES:

- It is better to clean all parts in contact with the breast milk immediately after using the breast pump. It facilitates the cleaning and avoids the development of bacteria.
- If condensation appears inside the tube or when milk enters inside, rinse the tube in order to remove liquid residues.
- The interface bottle prevents any liquid penetration into the breast pump. It is therefore important to clean it with soapy water and disinfect it in boiling water (10 minutes), as soon as it has been soiled. Check if the breast pump is not wet or damp. If it is, do not use it.
- Do not use the milk that penetrated into the interface bottle or tubes. The tubes should be briefly dipped into boiling water (10 minutes) and then thoroughly dried. The tubing colour may change but it will not alter its good functioning.

c) Disinfection at home

Accessories concerned:

- KOLOR® breast shield
- Collection container
- Cap

For full-term babies (without medical problems), disinfect before the first use and once a day. For premature babies or those with weakened immune systems, disinfect before the first use and after each use.

- ① Disinfect every part in boiling water (10 minutes), in a dishwasher or in a microwave steriliser.
(Recommendation for disinfection with microwave sterilizer: 800w-950w for 4 min)
- ② Let them dry in an open air on a paper towel or clean linen.

NOTES:

- In the case of intensive disinfection (premature infants or with a weakened immune system), boiling water is the only method that guarantees an optimal lifetime of the accessories.
- If the disinfection takes place in boiling water and in case of hard water, 2 or 3 drops of white vinegar can be added in the water to avoid whitish deposits on the parts.
- In the case of disinfection with a dishwasher, the plastic parts may be discoloured by food pigments. It does not affect their functioning.
- Regularly check the condition of your breast shields during use, cleaning and disinfection.
- If signs of degradation or color change appear, please replace the accessories.

	Breast shield	Collection container	Cap	Tubes & interface bottle	Breast pump
Before the first use and after each use					
Cleaning	✓	✓	✓	✗	✗
Before the first use and once a day					
Disinfection	✓	✓	✓	✗	✗
When it has been soiled					
Cleaning & disinfection	✗	✗	✗	✓	✗
Before passing on to the next mother					
Cleaning & disinfection (Cloth with disinfectant)	✗	✗	✗	✗	✓

d) Cleaning and sterilization by healthcare establishments

The autoclavable KIT EXPRESSION (containing an autoclavable KOLOR® breast shield, a collection container, a cap and a silicone tube) can be autoclaved.

Breast shields, caps, collection containers and silicone tubes have been designed to be sterilized at 125°C during 30 minutes or 134°C during 18 minutes. The total number of cycles should not exceed 50 cycles. Do not use the breast shield in case of damage or failure. The interface bottle is not autoclavable. It is therefore

important to discard it as soon as it has been soiled. For more information, please refer to the autoclavable breast shield instruction for use.

4. Use

Note: Follow the exact order of the steps below to get the correct use of the breast pump.

a) Breast pump functioning

The FISIO® PRO breast pump should be plugged into the power supply.

- Check that the values on the breast pump (volts and Hz) corresponds to the values of your distribution network.
- Connect the breast pump to the power supply.

NOTES:

- Check if the breast pump is suitable for the wall outlet.

b) Accessories preparation

NOTES:


- Only use the accessories provided in the kit or those manufactured by DTF MEDICAL.
- Before use, check that the KIT EXPRESSION components are free from defect, damage or sign of wear. If this is the case, replace them.
- All the components must be completely dry before use.

Before the first use, please refer to chapter 3 for cleaning and disinfection.

- ❶ Wash your hands with hot and soapy water before handling the breast pump parts.
- ❷ Screw the KOLOR® breast shield onto the collection container.
- ❸ Connect the suction tube to the inlet hose of the breast shield and to the interface bottle on the unit.

Set up of the single KIT EXPRESSION:

Set up of the double KIT EXPRESSION (can also be used as single one):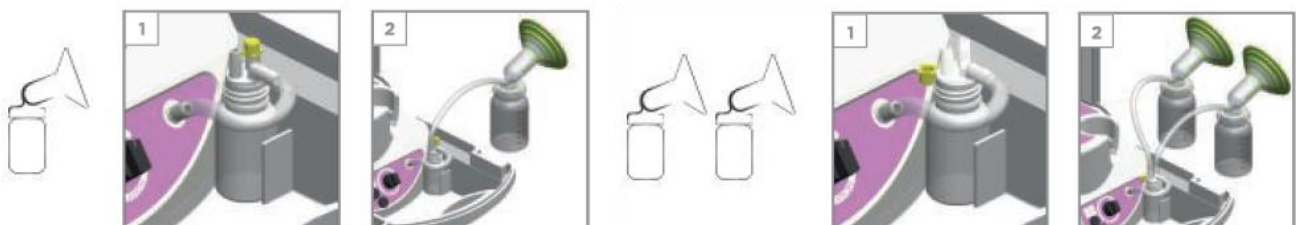


WARNING: Correctly close the interface bottle cap when the breast pump is used on simple pumping.

c) Using the breast pump

- ❶ Place the breast shield correctly over your breast ensuring the milk can flow freely into the collection container
- ❷ Ensure your nipple is centred inside the breast shield. The flexible part of the breast shield must also be properly positioned on your breast to ensure complete sealing.
- ❸ The cycle button allows adjusting the suction cycles. The suction button graduated from 1 to 9 allows adjusting the suction from minimum to maximum. To stimulate the lactation, set the suction button to 1 (lowest suction) and the cycle button to 120 (quick cycles).
- ❹ Switch on the breast pump (position "I").

It is necessary to start with this stimulation phase in order to stimulate the milk arrival and to activate the milk ejection reflex. This phase should last about 2 minutes. According to your own comfort level, you may adapt these settings by increasing the suction force and/or decreasing the cycle.

- ❺ After the ejection reflex, select progressively a slower cycle by turning the cycle button clockwise and increase the suction button to obtain a good expression according to your own level of comfort. Milk expression is optimal at the level of maximum depression and which should not be painful.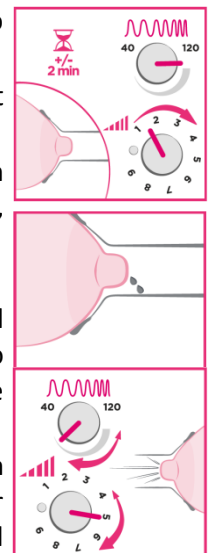

- ⑥ Check the proper flow of milk inside the collection container.
- ⑦ After expressing the milk, switch off the breast pump (position “O”).

NOTES:

- Always unplug the breast pump from the power supply after use.
- Do not fill the collection container beyond the marking.
- Be careful not to pinch the tubes during use.

d) After each use

- ① Disconnect the breast shield tube(s) and unscrew the breast shield(s) from the collection container(s).
- ② Secure the cap(s) onto the collection container(s)
- ③ Disconnect the tube(s) from the interface bottle. Check that no milk has penetrated into the interface bottle or inside the tubes.
- ④ After each use, rinse and clean every part as described in chapter 3.
- ⑤ Store the breast pump and its accessories in a dry place away from sunlight.

5. Breastfeeding

a) Storing breast milk

Information on the preservation of breast milk is available on the website www.dtf.fr and on the application KOLORYOU®.

Do not thaw or heat the collect milk in the microwave.

b) Baby's breastfeeding physiology

The FISIO® PRO breast pump provides mothers with the natural sensation of breastfeeding by recreating the way in which the baby feeds.

It reproduces the 2 physiological phases of the breastfeeding:

- The stimulation phase to generate the milk let-down (short, rapid movements),
- The expression phase to collect milk (longer, less frequent movements).

6. Safety precautions

Use of electricity

- Before connecting the breast pump to the power supply, check the values on the breast pump (Volt –V- and Hertz –Hz-) with your distribution network.
- The warranty does not cover faulty connections.
- This medical device is designed to operate at altitudes less than or equal to 3 000 meters.
- Do not open the breast pump when connected to the power supply.
- Warning: this medical device is directly connected to the electric power supply, do not use it in bath or under the shower, do not immerse.
- Keep this medical device away from any high heat source.
- Connect the breast pump to an easy-to-access socket to allow quick disconnection of the network in the event of a problem.
- The plug is the disconnection device and must always be accessible.

Use of breast pump

- This medical device must not be left within the reach of unsupervised children or animals (potential hazards with tubing and detachable components, risk of electrification).
- The breast pump should be installed and started up in accordance with the information supplied in the accompanying documents.
- Properly store the breast pump and its accessories after use.
- An animal may gnaw at the power cord. Be sure to check the condition of the equipment before use.
- Do not use the appliance outdoors, near a heat source or near a water source (bathtub, sink ...).
- Make sure the unit is not exposed to direct sunlight or excessive heat.
- This medical device must not be used in presence of a mixture of inflammable anaesthetic with air or oxygen or nitrous oxide.

- After use, unplug the appliance from the wall outlet.
- Fragile medical device, to be handled with care.
- Keep these instructions in a safe place with the appliance as it contains the necessary information for the proper use of the equipment.
- This medical device is to be used either in health care establishment or at home.
- The user can safely use all the functions of the breast pump.
- For hygiene reasons, a new complete KIT EXPRESSION must be sold to each mother.

WARNINGS:

- Do not touch the breast pump box for more than 1 second during use.
- The tube connecting the collection container to the breast pump can reach the temperature of 43.5°C. The user should not touch the tube for more than 10 minutes during the breast pump use.

Notes on the functionality of the breast pump

- The international version of the KOLOR® breast shield (with an international screw thread cap) can only be used with a matching collection container (same screw thread cap).
- Switch off the breast pump when the collection container is filled in order to prevent milk from getting into the interface bottle.
- If the device has to be rinsed, entrust the operation to a maintenance professional (risk of bad handling).

CAUTION, in all cases:

- WARNING: Always disconnect the plug from the power supply before opening it.
- WARNING: Only an approved customer service is authorized to repair the breast pump.
- WARNING: Modifications are forbidden.
- WARNING: Only use the accessories supplied and mentioned in the instructions for use.
- WARNING: The warranty does not cover connection errors with non-compatible accessories.
- WARNING: Do not undertake service or maintenance operations while the breast pump is used.


7. Problem solving

Problem	Solution
No operating noise can be heard from the pump	Make sure that the power cord is connected to the power supply. Check whether the switch is on the operating position. Check that the current delivered by the power supply corresponds to that of the breast pump (Volt Hz). Check with another breast pump if power supply is available in the wall outlet. If the problem continues, please contact DTF MEDICAL or your local distributor.
There is low or no suction	Check that the breast shield is fully screwed onto the collection container (screw tightly). Check that every tube is properly connected and not pinched. Check that the cap of interface bottle correctly fits. Check the condition of the O-ring between the breast shield and the collection container. Check that there is no residue in the tube. If the problem continues, please contact DTF MEDICAL or your local distributor.
The suction is too important	Switch off then switch on the breast pump. If the problem continues, please contact DTF MEDICAL or your local distributor.
The breast pump stops abruptly when used	Disconnect the tube from the interface bottle to vent it. In case of alternating stop, please contact DTF MEDICAL or your local distributor.
The pump sucks continuously	Do not tear off the breast shield from your breast (could cause pain and/or damage). Insert your finger between your breast and the breast shield to let the air come in and remove it more easily or disconnect the tube from the breast shield.
A piece or an accessory is missing	Please contact DTF MEDICAL or your local distributor.
The breast pump is too noisy or malfunctioning	Please contact DTF MEDICAL or your local distributor.
Milk entered inside the interface bottle	Clean it as soon as it has been soiled or discard it if you are in a healthcare establishment. Check if the breast pump is not wet or damp. If it is, do not use the breast pump. Do not use the milk that penetrated into the interface bottle or tubes. The tubes should be briefly dipped into boiling water and then thoroughly dried.

Problem	Solution
Milk was spilt or entered inside the breast pump	Switch off the breast pump. Unplug the breast pump for the power supply. Let the breast pump in a dry and warm place during 24 hours. Do not expose the breast pump to direct sunlight. Do not accelerate the drying process.

Please contact DTF MEDICAL or your local distributor:

- for assistance, if necessary, in the assembly, use or maintenance of the breast pump,
- to report any unexpected operation or event.

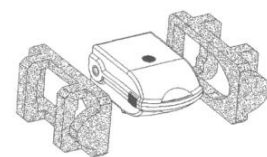WARNING: Keep the specific packaging and foam set for sending back the breast pump.

8. List of accessories and spare parts available

Part designation	Item code
KIT EXPRESSION	See below
Autoclavable KIT EXPRESSION	See below
KOLOR® breast shields	See below
Autoclavable KOLOR® breast shields	See below
FISIO® BIB – Batch of 2 collection containers 100mL (international screw thread)	FIBIBN-E
FISIO®COOL – Set of 2 cooling packs	FICOOOL
FISIO®FRESH – 4 collection containers, 4 cooling packs and 1 pouch	FIFRESH-E
PVC tube, 1 m-length, for KIT EXPRESSION	Y1091
Silicone tube, 1 m-length, for autoclavable KIT EXPRESSION	TU32
Plastic interface bottle, for double or single pumping	W2
Soft transport bag for the breast pump	MBOXK
Foam filter	YR36

KIT EXPRESSION:

Kit Ref.	Breast shield size	Shape size	Single / Double	Kit Ref.	Breast shield size	Shape size	Single / Double
K21ES	21mm	Small	Single	K24ES	24mm	Small	Single
K21EL	21mm	Large	Single	K24EL	24mm	Large	Single
K21ESD	21mm	Small	Double	K24ESD	24mm	Small	Double
K21ELD	21mm	Large	Double	K24ELD	24mm	Large	Double
K26ES	26mm	Small	Single	K30ES	30mm	Small	Single
K26EL	26mm	Large	Single	K30EL	30mm	Large	Single
K26ESD	26mm	Small	Double	K30ESD	30mm	Small	Double
K26ELD	26mm	Large	Double	K30ELD	30mm	Large	Double

Autoclavable KIT EXPRESSION (simple kit):

Kit Ref.	Breast shield size	Shape size	Kit Ref.	Breast shield size	Shape size
K21ESH	21mm	Small	K24ESH	24mm	Small
K26ELH	26mm	Large	K30ELH	30mm	Large

KOLOR® breast shield sold alone:

Kit Ref.	Breast shield size	Shape size	Kit Ref.	Breast shield size	Shape size
WB21ES	21mm	Small	WB24ES	24mm	Small
WB21EL	21mm	Large	WB24EL	24mm	Large
WB26ES	26mm	Small	WB30ES	30mm	Small
WB26EL	26mm	Large	WB30EL	30mm	Large

Autoclavable KOLOR® breast shield sold alone:

Kit Ref.	Breast shield size	Shape size	Kit Ref.	Breast shield size	Shape size
WB21ESH	21mm	Small	WB24ESH	24mm	Small
WB26ELH	26mm	Large	WB30ELH	30mm	Large

9. Warranty / Recycling

Warranty:

- The breast pump is guaranteed for 3 years from the date of purchase against any manufacturing defect, except case of misuse.
- For after sales service under warranty, please contact DTF MEDICAL or your local distributor.
- The technical documentation is available on request.

Recycling:

The device must be disposed of in accordance with local regulation on the disposal of electrical appliances. So you contribute to the human health and environmental protection. Product on the market since August the 13th 2005.

10. Technical specifications

Device class (breast pump)	IIa	
Device class in electrical safety (breast pump)	II	
Type of applied parts	BF	
Protection level	IP 30	
Expected shelf life of accessories	1 year	
Expected shelf life of autoclavable accessories	50 cycles	
Expected shelf life of the breast pump	5 years	
Collection container capacity	100ml	
Weight	4 kg	
Dimensions	L360 x l250 x h180mm	
Voltage	Available in 230V – 50Hz; 220V – 50Hz or 220V – 60Hz	
Consumption	100VA	
Fuses	T 0,5A H 250V	
Suction values	Variable from -50 to -300hPa	
Cycles	40 to 125 cycles/minutes	
Sound level	<57dBa	
Running mode	Continuous	
Power supply	Alternative current 230V – 50Hz	
Ambient temperature (T) - For transport and storage		-25°C < T < +70°C
Ambient temperature (T) - When in functioning		+5°C < T < +40°C
Relative humidity (H) For transport, storage and functioning		15% < H < 93%
Atmospheric pressure (P) For transport, storage and functioning		700 hPa < P < 1060 hPa

Essential performance features:

- Provide adequate suction so that mothers can express their milk.
- Control the rate of suction to best meet the baby's sucking rhythm.

Applied part: Breast shield.

Identification:

Each breast pump has a unit identification (reference and number) which indicates, among other things, the exact date of manufacture and the serial number of the breast pump. The first two digits show the year of manufacture (e.g. **68**: 1950 + **68** = 2018) and the next three digits indicate the day of the year of manufacture (e.g. 285 = 12th October). The last two digits indicate the serial order number in the series. In addition, each accessory is identified by a lot number mentioned. For more information, refer to the instructions for use of the KIT EXPRESSION.

11. Electromagnetic compatibility (EMC)

Electro medical devices require special precautions against EMC and must be installed and set to operate in accordance with the EMC information provided in this manual.

WARNING: Avoid using this unit near or stacked onto other equipment as this may cause a malfunction. If this is necessary, this appliance and the other appliances should be monitored for normal operation.

WARNING: Use of accessories, transducers and cables other than those specified or supplied by the manufacturer of this device may cause an increase in electromagnetic emissions or a decrease in the immunity of this device and cause improper operation.

WARNING: Portable RF communication devices (including peripherals such as antenna cables and external antennas) should not be used closer than 12 inches (30 cm) to any part of the FISIO® PRO breast pump, including cables specified by the manufacturer. Otherwise, performance may be impaired.

WARNING: If the device malfunctions due to electromagnetic disturbances, the performance of the device may be impaired and result in delayed patient care.

Wireless communication devices such as wireless home networking devices, mobile phones, cordless phones and their bases, walkie-talkies can affect the device, and it is recommended to keep them at a distance of at least 0.5 m from the device.

Guidance and manufacturer's declaration – electromagnetic emissions	
Emissions test	Compliance
RF emissions (CISPR 11)	Group 1
RF emissions (CISPR 11)	Class B
Harmonic emissions (IEC 61000-3-2)	Class A
Voltage fluctuations / flicker emissions (IEC 61000-3-3)	Complies

Guidance and manufacturer's declaration – electromagnetic immunity		
Immunity test	IEC 60601 test level	Compliance level
Electrostatic discharge (ESD) (IEC 61000-4-2)	± 8 kV contact ± 15 kV air	± 8 kV ± 15 kV
Electrical fast transient / burst (IEC 61000-4-4)	± 2 kV for power supply lines ± 1 kV for input/output lines	± 2 kV ± 1 kV
Surge (IEC 61000-4-5)	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth
Voltage dips, short interruptions and voltage variations on power supply input lines (IEC 61000-4-11)	0% U _T ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°	0% U _T ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°
	0% U _T ; 1 cycle AND 70% U _T ; 25/30 cycles Single phase: at 0°	0% U _T ; 1 cycle AND 70% U _T ; 25/30 cycles Single phase: at 0°
Voltage interruptions (IEC 61000-4-11)	0% U _T ; 250/300 cycles	0% U _T ; 250/300 cycles
Power frequency (50/60 Hz) magnetic field (IEC 61000-4-8)	30 A/m	30 A/m
Radiated RF (IEC 61000-4-3)	10 V/m 80 MHz to 2.7 GHz	10 V/m 80 MHz to 2.7 GHz
	385 MHz at 27 V/m 450 MHz at 28 V/m 710, 745, 780 MHz at 9 V/m 810, 870, 930 MHz at 28 V/m 1720, 1845, 1970 MHz at 28 V/m 2450 MHz at 28 V/m 5240, 5500, 5785 MHz at 9 V/m	385 MHz at 27 V/m 450 MHz at 28 V/m 710, 745, 780 MHz at 9 V/m 810, 870, 930 MHz at 28 V/m 1720, 1845, 1970 MHz at 28 V/m 2450 MHz at 28 V/m 5240, 5500, 5785 MHz at 9 V/m
Conducted RF (IEC 61000-4-6)	3V 0,15 MHz – 80 MHz 6V in ISM bands and amateur radio bands between 0.15 MHz and 80 MHz 80% MA at 1kHz	3V 0,15 MHz – 80 MHz 6V in ISM bands and amateur radio bands between 0.15 MHz and 80 MHz 80% MA at 1kHz

12. Pictograms

Symbol on the breast pump, its accessories and on the packaging

In accordance with Directive 93/42/EEC of 14th June 1993 regarding medical devices

Manufacturer

Reference

Serial number (breast pump)

Batch number (accessories)

Device class II

IP30

Level of protection:

3: protected against ingress of solid foreign objects with a diameter ≥ 2.5 mm

0: not protected (against water penetration)

Applied part type BF

Alternative current

Follow instructions for use

Warning: Dangerous voltage

Caution/Warning

Keep dry; Fear of moisture

Fragile; Handle with Care

Store away from light

This symbol indicates the limit of atmospheric pressure not to be exceeded during operation, transport or storage.

This symbol indicates the moisture limit which must not be exceeded in the case of operation, transport, and storage.

This symbol indicates the temperature limit which must not be exceeded in the case of operation, transport, and storage.

«Off» (power off)

«On» (power on)

Do not throw away. Product on the market since 13th August 2005.

BPA free

International screw thread

Electric breast pump with intermittent medium suction and low flow.

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