Humidifier/Heater Model DV5HH

EN DeVilbiss® Heated Humidifier

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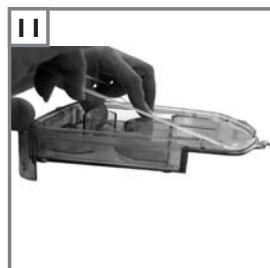
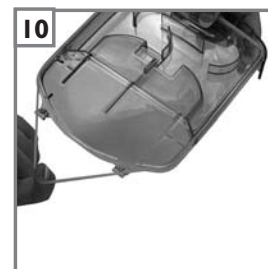
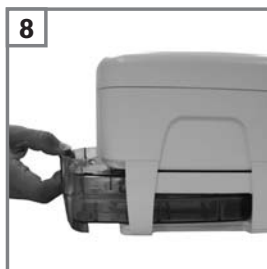
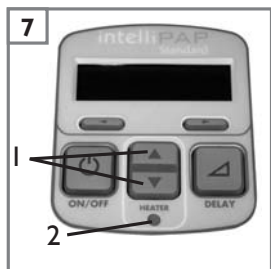
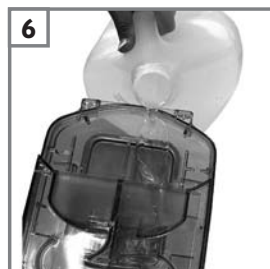
Set Up 5

 Unpacking the Contents 5

 Important Parts of Your Integrated Humidifier/Heater 5

 Replacement Items..... 6

 System Assembly..... 6

 Attaching Flow Generator..... 6

 Filling the Chamber 6

Operating Instructions..... 7

 Heater Controls 7

 Controlling the Humidity 7

 Daily Use Instructions 7

Cleaning Instructions 7

 Water Chamber 7

 Humidifier Cradle..... 8

Troubleshooting..... 8

Service Instructions 8

Specifications 9

Warranty..... 9

Declaration of Conformity 10

INTRODUCTION

The Heated Humidifier system is an optional accessory to the IntelliPAP and SleepCube series flow generators. It is intended to provide heated humidity to improve the comfort and outcomes of positive airway pressure therapy.

Indications For Use

Use on the advice and prescription of a licensed physician to help relieve the symptoms of dryness of the throat, nasal passages and the mouth, which are common with positive airway pressure therapy. This is especially true in dry climates and during the cold season when humidity in the air is typically lower than at other times.

Contraindications

Do not use the Heater Humidifier if your upper airway has been bypassed.

SYMBOL DEFINITIONS

	This device contains electrical and/or electronic equipment that must be recycled per EC Directive 2002/96/EC - Waste Electrical and Electronic Equipment (WEEE)		Decrease value shown on LCD display
	Type BF equipment— applied part		Attention - Consult Accompanying Documents
	Class II electrical protection— double insulated		Heat
	Standby – turns the blower and heater on or off		100 – 240 Volts AC input 50/60 Hz
	Increase value shown on LCD display		IPX1 Drip Proof, Vertical

IMPORTANT SAFEGUARDS

Please read all instructions carefully before use. The information in this manual will help you to understand the operation of the humidifier and how to properly maintain it. Important information is highlighted by the following terms:

DANGER Urgent safety information for hazards that will cause serious injury or death.

WARNING Important safety information for hazards that might cause serious injury.

CAUTION Information for preventing damage to the product.

NOTE Information to which you should pay special attention.

PLEASE READ ALL INSTRUCTIONS BEFORE USING THIS DEVICE.

SAVE THESE INSTRUCTIONS

DANGER

Electric Shock Hazard – Do not use while bathing.

Electric Shock Hazard – Do not immerse this device into water or any other liquid.

Electric Shock Hazard – Do not attempt to open or remove the enclosure. There are no user-serviceable internal components. If service is required, return the product to your home care provider. Opening or tampering with the product will void the warranty.

WARNING

Use the Heated Humidifier system only for its intended use as described in this manual.

Be sure to read and understand all safety instructions supplied with your IntelliPAP or SleepCube flow generator device.

Use only accessories recommended by DeVilbiss.

Always remove the water chamber from the humidifier cradle for filling.

Use only tubing recommended or supplied with the flow generator.

If water has been spilled onto the cradle or it has been submerged into water, unplug the power cord from the power source immediately. Allow the device to completely dry before use.

If the device has been dropped, refer to the troubleshooting guide for instructions. Contact your homecare provider for evaluation of the device.

Never block the air openings of the cradle or chamber. Do not insert any objects into any openings or tubes.

For proper operation of the humidifier, the device must be placed on a flat, level surface.

Never place the system on a soft surface such as a bed or couch during operation.

This heated humidifier is intended for single-patient use only.

Be sure that you read and understand the information in this instruction guide. By following these instructions and the advice of your physician, your heated humidifier system will become an effective addition to your CPAP therapy.

Empty and dry the water chamber before transporting.

CAUTION—The height of the system must always be lower than the interface mask during use to prevent water from getting into the CPAP mask.

SET UP

Unpacking The Contents

The following items are included in your Heated Humidifier system:

1. Humidifier Cradle with Water Chamber (Figure A)
2. User Manual

NOTE—Please contact your home care provider if any of the above items are missing.

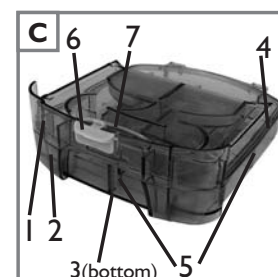
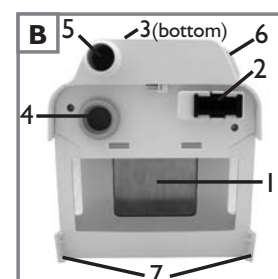
Important Parts Of Your Integrated Humidifier/Heater

Humidifier Cradle (Figure B)

1. Heater plate
2. Heater power connector
3. Storage compartment for heater power connector soft cover (on bottom)
4. Air supply port inlet
5. Air supply port outlet
6. Flow generator release button
7. Flow generator locking tabs

Water Chamber (Figure C)

1. Chamber lid
2. Chamber base
3. Heat transfer plate (on bottom)
4. Sealing gasket
5. Water level indicators (front and sides)
6. Chamber release latch
7. Chamber disassembly lever

Replacement Items

<u>Item</u>	<u>Part Number</u>
Humidification System.....	DV5HH
Humidifier Chamber	DV5C
Chamber Sealing Gasket	DV5C-614

System Assembly

NOTE—Follow Steps 1-3 if you received your humidifier system and CPAP separately. Start with Step 4 if you received the complete system in one carton.

1. Remove the air supply tubing from the back of the device.
2. Locate the air supply port at the bottom of the device. Remove the port plug and insert it into the air supply port in the back of the device (Fig 2).
3. Locate the heater power connector on the bottom of the device. Remove the connector cover and insert it into the storage opening in the bottom of the humidifier cradle (Fig 1).

Attaching Flow Generator

4. For easier attachment, remove the water chamber before assembling the flow generator and humidifier cradle.
5. Align the base of the device with the cradle at a slight angle as shown (Fig 3). Slide the device forward and align cradle locking tabs with the recesses in the CPAP. Firmly press down the back to lock into place.
6. Verify that both sides of the device have engaged into place with the humidifier cradle.
7. Attach the air supply tubing to the air supply port outlet on the humidifier cradle (Fig 4).
8. Insert the power cord into the back of the flow generator. Make sure the power cord is fully inserted into the connector (Fig 5). Plug the other end of the power cord into the AC wall outlet.

CAUTION—The height of the system must always be lower than the interface mask during use to prevent water from getting into the CPAP mask.

Filling The Chamber

1. Hold the chamber upright and fill with distilled water to the top of the opening (Fig 6).

CAUTION—Use only distilled water that is at room temperature. Do not add any medication or other additives to the water.

2. Carefully tip the chamber back to the level position. The water in the fill ports will flow into the chamber when it is tipped back to the level position.

CAUTION—For proper operation, ensure the flow generator is OFF before inserting humidifier chamber.

3. Slide the chamber back into the humidifier cradle. Ensure the chamber is fully engaged with the humidifier cradle and the chamber release latch snaps in place when inserted.

WARNING

Do not attempt to fill the humidifier while it is attached to the humidifier cradle. Damage to the humidifier cradle may occur.

If the system is placed on the floor, do not position it where it may be stepped on when getting out of bed.

Never touch the heater plate on the humidifier cradle. Never touch the heat transfer plate on the bottom of the water chamber. These plates can reach temperatures as high as 149°F (65°C) during operation.

Do not operate the heater if the water chamber is empty. The heater plate may be turned off using the keypad heater control when being used without water.

OPERATING INSTRUCTIONS

Heater Controls

The heater controls and status are located on the keypad and display.

1. Heater Setting Control Buttons
2. Heater Power LED

Controlling The Humidity

The heater setting can be adjusted in a range of 1 to 10. The optimum temperature setting is dependent on the ambient temperature and humidity of your room. If your room temperature is low and/or the relative humidity is high, then a lower temperature setting may be necessary to prevent excess condensation in the air supply tubing. If the room temperature is high and/or the relative humidity is low, then a higher temperature setting may be necessary for optimum comfort.

The heater setting of 1 results in a heater plate temperature of approximately 84°F (29°C). The heater setting of 10 results in a heater plate temperature of approximately 149°F (65°C).

Daily Use Instructions

NOTE—Always allow at least 10 minutes of cooling time after turning the device off before removing the water chamber from the humidifier cradle.

1. Press down on the chamber release latch of the humidifier and remove the chamber from the cradle (Fig 8).
2. Open the chamber and empty any water remaining from the previous night.
3. Clean the chamber according to the cleaning instructions and reassemble.

CLEANING INSTRUCTIONS

Water Chamber

1. Press down on the chamber release latch and pull out the water chamber from the cradle (Fig 8).
2. Separate the chamber halves by pressing forward on the chamber disassembly lever and lifting the lid of the chamber (Fig 9).
3. Remove the sealing gasket from the chamber lid (Fig 10).
4. Wash the parts in a solution of warm water and mild dishwashing detergent.

CAUTION—Hand wash only. Do not use any solutions that contain bleach, chlorine, alcohol, scented oils or other aromatic-based ingredients to clean the humidifier components or tubing. These solutions may reduce the life of the product.

5. Allow the parts to dry before reassembly.

Reassembly

1. Insert the gasket seal into the groove on the chamber lid. Inspect the fitting of the gasket to ensure it is fully in place (Fig 11).
2. Snap the chamber base and lid together by inserting the tabs on the lid into the slots on the base.

CAUTION—Inspect the gasket seal before each use to ensure that it is properly seated in the groove around the entire chamber.

Humidifier Cradle

NOTE—The humidifier cradle should be cleaned as needed.

1. Turn off the heater and unplug from the power source. Allow the system to cool for at least 10 minutes.
2. To detach the flow generator from the humidifier cradle, first remove the water chamber then fully depress the flow generator release button to separate (Fig 12).

NOTE—The flow generator release button must be pressed completely to separate.

3. Use a soft damp cloth to clean the surfaces of the unit.

TROUBLESHOOTING

Problem	Probable Cause	Remedy
Unit does not turn on. Power indicator not lit.	1. Flow generator is not plugged in or the power cord is not fully inserted. 2. The flow generator is not fully seated onto the humidifier cradle. 3. There is no outlet power.	1. Plug the power cord into a live power source. Ensure that the power cord is properly connected to the unit and the power source. 2. Refer to System Assembly in this manual for instructions on system setup. 3. Locate a suitable functioning power source.
The air supply does not seem to be high enough.	The flow generator is not fully seated onto the humidifier cradle.	1. Refer to System Assembly in this manual for instructions on system setup. 2. Make sure the air supply port plug is fully inserted into the back of the CPAP.
The device reports a mask leak but my mask seems to be fitting well.	1. The humidifier chamber is not fully engaged or is missing. 2. The air supply port plug is missing from the back of the CPAP or not fully inserted.	1. Slide the humidifier chamber fully into the cradle. Verify the release latch clicks into place. 2. Make sure the air supply port plug is fully inserted into the back of the CPAP. Refer to System Assembly in this manual for instructions on system setup.
The heater setting is not visible on the LCD display.	The flow generator is not fully seated onto the humidifier cradle.	1. Refer to instructions in System Assembly. If the problem continues, contact your home care provider.
Dryness of throat or nose.	Inadequate humidity level.	Increase the heater setting on the humidifier.
Excessive condensation or water in tubing.	1. Heater control setting too high. 2. Heated humidifier is placed higher than patient. 3. Humidifier chamber overfilled.	1. Reduce heater control setting. 2. Move unit to a lower position. 3. Water level in chamber should not be above the maximum level.

SERVICE INSTRUCTIONS

The humidifier cradle does not require regular servicing.

SPECIFICATIONS

Humidity Output (in the operating flow range):	≥10 mgH ₂ O/l air
Size:	2.6" H x 6.3" W x 8.4" D (6.6 cm x 16.0 cm x 21.3 cm)
Weight:	1.75 lbs. (0.794 Kg)
Electrical Supply Frequency:	50/60 Hz
Power Consumption:	85 Watts
DV5HH	100-240VAC, 0.95 A
Heater Plate Thermal Cutout:	105°C (221°F)
Power/Temperature Control:	Setting of I (minimum) to 10 (maximum)
Heater Plate Temperature:	approx. 29°C to approx. 65°C (84°F to 149°F)
Operating Temperature Range:	41°F to 104°F (5°C to 40°C)
Operating Humidity Range:	0 to 95% R.H. non-condensing
Operating Atmospheric Pressure Range:	70.0 – 106.0 kPa
Transport & Storage Temperature Range:	-40°F to 158°F (-40°C to +70°C)
Transport & Storage Humidity Range:	0 to 95% R.H. non-condensing
Transport & Storage Atmospheric Pressure Range:	50.0 – 106.0 kPa

Class II Equipment; Type BF Applied Parts; Continuous Operation
IPX1, Drip-proof vertical

TWO-YEAR LIMITED WARRANTY

The DeVilbiss Heated Humidifier System Model DV5HH is warranted to be free from defective workmanship and materials for a period of two years from date of purchase (except 90 days on chamber). Any defective part(s) will be repaired or replaced at DeVilbiss's option if the unit has not been tampered with or used improperly during that period. Make certain that any malfunction is not due to inadequate cleaning or failure to follow the instructions. This warranty does not cover normal wear and tear on the o-ring seal. If repair is necessary, contact your DeVilbiss provider or DeVilbiss Customer Service Department for instructions:

U.S.A 800-338-1988 (814-443-4881)
Canada 905-660-2459
Europe 44-138-444-6688

NOTE—This warranty does not cover providing a loaner unit, compensating for costs incurred in rental while said unit is under repair, or costs for labor incurred in repairing or replacing defective part(s).

THERE IS NO OTHER EXPRESS WARRANTY. IMPLIED WARRANTIES, INCLUDING THOSE OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE EXPRESS LIMITED WARRANTY AND TO THE EXTENT PERMITTED BY LAW ANY AND ALL IMPLIED WARRANTIES ARE EXCLUDED. THIS IS THE EXCLUSIVE REMEDY AND LIABILITY FOR CONSEQUENTIAL AND INCIDENTAL DAMAGES UNDER ANY AND ALL WARRANTIES ARE EXCLUDED TO THE EXTENT EXCLUSION IS PERMITTED BY LAW. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, OR THE LIMITATION OR EXCLUSION OF CONSEQUENTIAL OR INCIDENTAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

DECLARATION OF CONFORMITY

Manufacturer: Sunrise Medical dba DeVilbiss Healthcare
Address: 100 DeVilbiss Drive
Somerset, Pennsylvania 15501-2125 USA

Product Designation: CPAP

Type, Model: DeVilbiss® IntelliPAP™ or SleepCube™ DV5x Series CPAP Models DV51 Standard CPAP and DV54 AutoAdjust and DeVilbiss Heated Humidifier Model DV5HH

We herewith declare that the above-mentioned product complies with the requirements of EC Directive 93/42/EEC and the following:

Class: IIa, Rule 2

Quality System Standards Applied: ISO 13485:2003

Notified Body: TÜV NORD

MDD: Annex II Applied

Safety Standards Applied: IEC 60601-1:1988 + A1 1995
IEC 60601-1-4:2000
ISO 14971:2000
EN ISO 17510:2002.

EMC Compliance To: IEC 60601-1-2:2001

Authorized Representative: Sunrise Medical Ltd.
Sunrise Business Park
High Street
Wollaston, West Midlands DY8 4PS ENGLAND
44-138-444-6688

®Registered U.S. Patent and Trademark Office and other countries.

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®Registrado en la Oficina de patentes y marcas (Patent and Trademark Office) estadounidense y en otros países.
®Gedeponeerd bij het U. S. Patent and Trademark Office en in andere landen.
Geautoriseerde vertegenwoordiger: Sunrise Medical Ltd.