RC-CORNET[®] PLUS NASAL / RC-CORNET[®] PLUS NASAL SET

DE	Gebrauchsanweisung	Seite	2
EN	Instructions for use	Page	14
ES	Instrucciones de uso	Página	26
IT	Istruzioni per l'uso	Pagina	38
FR	Notice d'utilisation	Page	50

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THANK YOU FOR CHOOSING THE RC-CORNET® PLUS NASAL THERAPY.

Additional Instructions for Use

COMBINED THERAPY with nebulizers using enclosed elbow connector see page 22 and 23

RC-CORNET® PLUS NASAL SET for converting RC-Cor-net® PLUS into RC-Cor-net® PLUS NASAL see page 25

NOTE

Please read the instructions for use carefully before using RC-Cor-net® PLUS NASAL for the first time.

If you have any questions regarding the use of RC-Cor-net® PLUS NASAL, please contact your healthcare professional. RC-Cor-net® PLUS NASAL is designed for patients age 2.5 and older. The patient must be able to consciously coordinate inhalation and exhalation by mouth. Children should only use the device under adult supervision. Prior to use, patients with cystic fibrosis should discuss use of the device with their healthcare professional.

The RC-Cor-net® PLUS NASAL tube is made of silicone. Please replace the tube every six months. We recommend the exchange of the RC-Cor-net® PLUS NASAL after one year of using. Whitish deposits on the tube or in the body are medical talc used during tube manufacture.

Store RC-Cor-net® PLUS NASAL in a dry, dust-free environment and protect it from sunlight.

Disposal: RC-Cor-net® PLUS NASAL and its accessories can be disposed of as normal household waste provided a different form of disposal is not mandated by national waste disposal regulations.

⚠ WARNING

If the patient is allergic to medical talc, please boil all RC-Cor-net® PLUS NASAL parts twice before using them for the first time. Otherwise please clean prior to first use following the cleaning instructions (see page 20 and 21).

Note: Do not use dishwashing detergent.


Please ensure that RC-Cor-net® PLUS NASAL is used properly according to the Instructions for Use. Incorrect application (off-label use) results in exclusion of liability for the manufacturer.

Please discontinue the therapy and contact your healthcare professional, if you feel dizzy, lightheaded, or unwell.

RC-CORNET® PLUS NASAL

For physical therapy of upper respiratory tract diseases as well as the middle ear using oscillatory positive expiratory pressure (OPEP) therapy.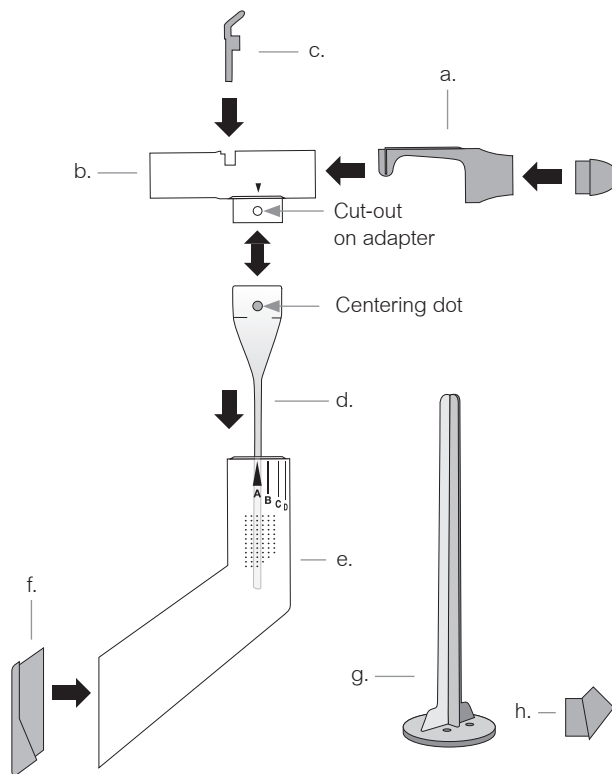

The RC-Cor-net® PLUS NASAL package contains:

- | | |
|---|---|
| a. Nosepiece with nozzle | f. Body lid |
| b. Adapter with integrated nebulizer port | g. Cleaning aid |
| c. One way valve | h. Elbow connector for combined therapy with nebulizers (Fig. see page 22 and 23) |
| d. Tube | |
| e. Body | |

Materials:

Silicone, Thermoplast and Polypropylene

INDICATIONS FOR USE

RC-Cornet® PLUS NASAL is a medical device to mobilize and remove secretions in the nasopharynx. It also improves aeration of the Eustachian tube in the event of disorders of the auditory tubes. Regular use provides a clear nose in a natural way in case of chronic sinusitis, post-nasal-drip syndrome and nose spray addiction (privismus). Further more bronchial hyperactivity is decreased in asthma bronchial.

Effectiveness of RC-Cornet® PLUS NASAL Therapy

- Frees the upper airways from irritating phlegm
- Reduces coughing due to post-nasal-drip
- Reduces bronchial hyperreactivity at asthma
- Improves aeration of the sinuses and middle ear

RC-Cornet® PLUS NASAL can also be combined with a nebulizer. This optimizes aerosol deposition and reduces therapy time (combined therapy - see page 22 and 23).

Physiological effects

RC-Cornet® PLUS NASAL produces pressure variations (oscillations) in the upper respiratory tract and widens it. It also mobilizes and liquefies nasal secretions and widens / opens the Eustachian tube. Secretions are then easier to swallow or blow out.

Contraindications:

RC-Cornet® PLUS NASAL should not be used in patients diagnosed with any of the following conditions unless instructed by a doctor:

- after surgery or injuries of the mouth, face, skull or esophagus
- repeated, massive nosebleed
- in the presence of purulent discharge
- barotrauma
- untreated pneumothorax

In case of acute sinusitis or middle ear infection, RC-Cornet® PLUS NASAL should only be used after having consulted a doctor and the use of antibiotics. Please also check for contraindications with the medications used as well as your inhalation system if RC-Cornet® PLUS NASAL is used with combined therapy with nebulizer.

Side effects

No undesirable effects are known in conjunction with RC-Cornet® PLUS NASAL therapy. However, patients may develop "a stuffy-nose" when beginning treatment. This is a regulatory phenomena.

Note

To lower the noise level of the therapy, close off the body holes with your hand.



Use of RC-Cornet® PLUS NASAL does not rely on gravity. The device may be used in an upright, seated or standing position.

After you become accustomed to RC-Cornet® PLUS NASAL, try to improve your breathing technique. Breathe in through the nose, if possible, and after a brief pause, exhale through RC-Cornet® PLUS NASAL.

Important: RC-Cornet® PLUS NASAL uses your full expired air volume to produce vibrations and oscillations. Therefore, please expel all of your breath through RC-Cornet® PLUS NASAL in order to maximize your therapy outcome.



With strong polyp formations a progress in therapy is not guaranteed.

VISUAL GUIDE for use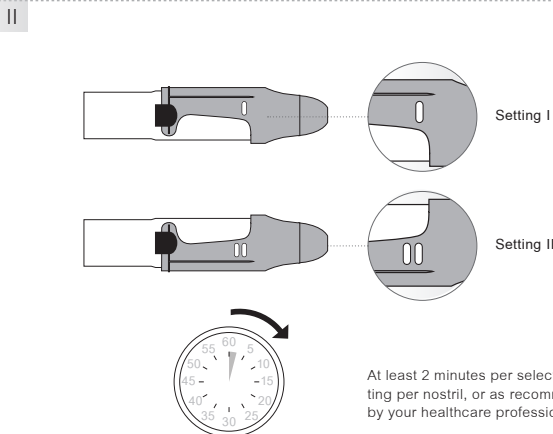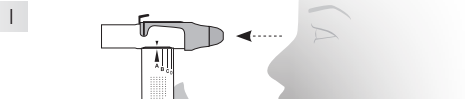

III

Inhale and exhale through RC-Cornet® PLUS NASAL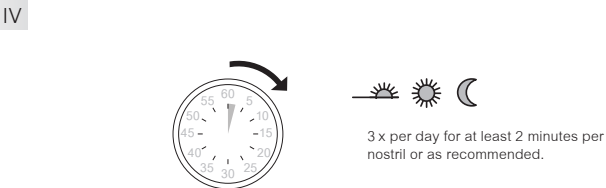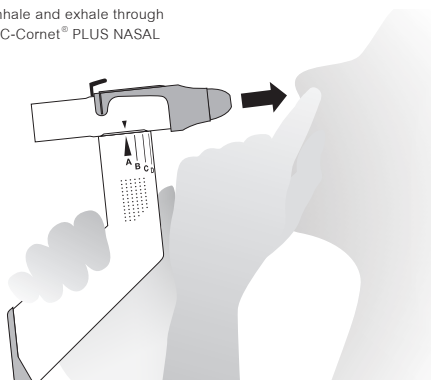

INSTRUCTIONS for use

I Please inspect the device for foreign bodies or damage prior to use. If any parts are missing or damaged, replace them immediately.

II **Settings:** RC-Cornet® PLUS NASAL can be used for two different forms of treatment. Select the treatment options using Setting I or Setting II. Change the settings by turning the nosepiece. For the middle position, center the nosepiece on the adapter. Unless otherwise recommended by your healthcare professional, use one setting for at least two minutes per nostril.

- **Setting I** stands for an oscillating positive expiratory pressure (OPEP). It expands the airways and relieves symptoms, including in the case of Eustachian tube dysfunctions. In addition, this setting improves the deposition of a possible saline solution or drug administration (see p. 11).
- **Setting II** stands for a mobilising OPEP. It frees the airways from thick nasal mucus. As a result, sniffing through the nose becomes more “productive”. Once the airways have been cleared, coughing due to post nasal drip syndrome and tension headaches diminishes automatically and the nose clears up.

III Hold the RC-Cornet® PLUS NASAL as shown. Insert RC-Cornet® PLUS NASAL into the right nostril and pinch the left nostril shut using your index finger. Breathe in through the nose, if possible, and after a brief pause, exhale through RC-Cornet® PLUS NASAL.


Should you experience difficulties, you can always remove your index finger and inhale over the left nostril. Breathe in through the nose, if possible, and exhale slowly through the nose and into RC-Cornet® PLUS NASAL. This produces a positive pressure as well as vibrations in the nasopharynx.

Exhale the full volume of air through RC-Cornet® PLUS NASAL. Repeat the process with the left nostril.

IV The recommended usage is at least 3 times daily for at least 2 minutes per nostril or as recommended by your healthcare professional. Additional usage is recommended when notice mucus obstruction. As with all types of physical therapy, RC-Cornet® PLUS NASAL is most effective when used regularly.

TIP: Make the therapy part of your daily routine, even when you are not congested, since mucus is generated consistently.

VISUAL GUIDE Cleaning

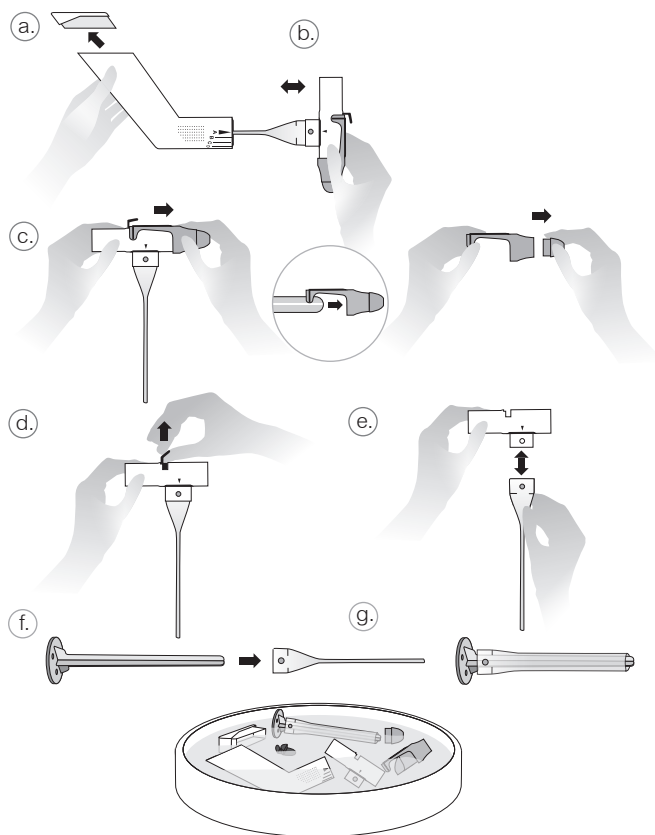INSTRUCTIONS Cleaning

Cleaning

Please wash your hands before cleaning. Take the entire RC-Cornet® PLUS NASAL apart as follows to clean the device.

The following order is manatory.

- Remove the body lid.
- Remove the adapter from the body.
- Remove the nosepiece from the adapter and remove the nozzle from the nosepiece, if necessary use the support of the cleaning aid.
- Pull the one way valve out of the adapter.
- Press together the upper end of the tube and pull the tube off the adapter
- f.+g. To clean, insert the cleaning aid into the valve tube.

Rinse all parts in warm water and/or add a few drops of citric concentrate/vinegar essence to the water. Rinse thoroughly several times with clean water.

Note: Do not use dishwashing detergent!

All parts may be cleaned in a vaporizer or by means of steam or in boiling water. Maximum temperature 134°C.

Further cleaning options with the RC-Clean® cleaning bag (see page 24).

Clinical recycling instructions can be found at www.cegla.de

Important! The cleaning aid must remain in the tube during the entire cleaning process. This prevents the walls of the tube from sticking together and guarantees an optimum cleaning process.

If used regularly, clean the device at least twice weekly.

Drying

Please remove all parts immediately from the cleaning medium (pot, vaporizer, RC-Clean®, etc.). Shake off any excess water and let all parts air dry. During the drying process, the cleaning aid is designed to hold the tube open. Place the tube with the cleaning aid in an upright position to reduce drying time. A complete drying reduces the risk of infections.

RECOMMENDED PRODUCTS


RC-Pflege® N

RC-Gel NASAL

Pursuant to the DEGAM-guideline (sinusitis), we recommend to clean the nose with a nasal cleanser prior to using RC-Cornet® PLUS NASAL. You also can use RC-Pflege® N nose spray. In our experience, this improves the therapeutic effect. Nasal douche should not be done if the nostril is inflamed or injured. If infected, inflamed or injured, use of RC-Gel NASAL is recommended.

VISUAL GUIDE Assembly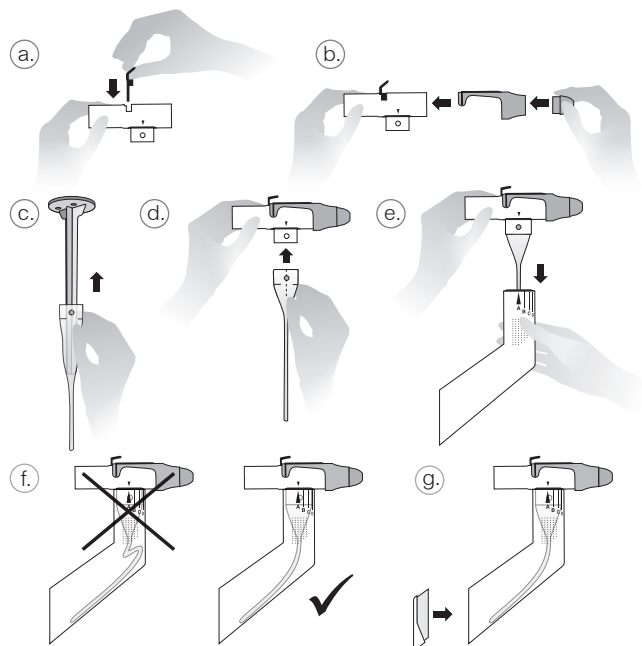


INSTRUCTIONS Assembly

Please wash your hands before assembly.

- Connect the one way valve with the adapter.
- Connect the nozzle to the nosepiece.
Center and connect the part to the adapter. For using the device please follow the instructions (see pages 18 and 19).
- Remove the cleaning aid from the tube.
- Connect the tube with the adapter and make sure that the centering dot clicks into the cut-out of the adapter.
- Guide the piece into the body. Make sure that the arrow on the adapter and on the body are aligned.
- Check the tube position. It should not be kinked.
- Reattach the body lid.

COMBINED THERAPY WITH NEBULIZER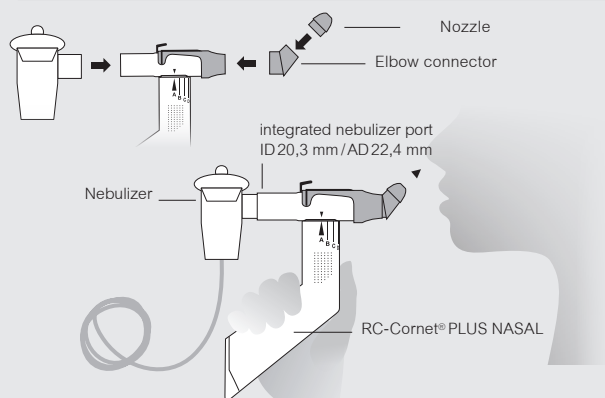

PLEASE NOTE:

In combination with nebulizers, it is important to ensure that the RC-Cornet® PLUS NASAL with and without the RC-Cornet® PLUS NEBULIZER SET is as securely fitted as it would be if the nebulizer mouthpiece was attached. In other words, during inhalation you should not see any inhalation solution flowing outside of the product combination. To this end, pay attention to the respective interior diameter (ID). Also, please read the instructions for use of your nebulizer and your inhalation device, as well as of the corresponding medication.

Combining RC-Cornet® PLUS NASAL therapy with a standard ISO nebulizer improves delivery of your nebulized medication and reduces the total treatment duration. Furthermore the effectiveness of both therapies is improved.


Insert the elbow connector between the nosepiece and nozzle. Connect RC-Cornet® PLUS NASAL to your nebulizer. To do so, first attach the nebulizer in a straight direction to the adapter. To detach the nebulizer from the adapter, pull it straight out and off.

IMPORTANT: Please do not “shake” the nebulizer. This may damage the device. Please always use the position I on RC-Cornet® PLUS NASAL for the combined therapy with nebulizers. This setting opens your airways and stabilizes them. Please inhale and exhale only through the device. The treatment duration corresponds to the inhalation time.

NOTE: The NEBULIZER SET is available for non ISO nebulizers. The nebulizer and the NEBULIZER SET are not included. They can be purchased separately. For more information visit www.cegla.de

RC-Cornet® PLUS NEBULIZER SET *

Article no. 2651 | PPN 111 443 990 049

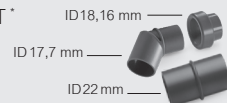* Depending on the application deviations in the deposition data are possible.

ACCESSORIES AND ADDITIONAL PRODUCTS

RC-Cornet® PLUS valve tubes

Applicable for all product variants
of RC-Cornet® PLUS.

Article no. 2660 | PPN 111 305 819 488

RC-Clean®

Reusable microwaveable cleaning bag.

Easy and fast cleaning of medical products
as well as infant items.

Use only for microwave safe products.

Article no. 5000 | PPN 111 075 103 265

Accessories and related products are available at pharmacies and medical supply stores.

RC-Cornet® PLUS NASAL is intended for single-patient use only. Instructions for clinical reuse for multiple patients are available at www.cegla.de

PPN 111 244 133 055

In an effort to improve our level of service, we welcome your feedback. Regular therapy with the RC-Cornet® PLUS NASAL leads to a better quality of life.

All trademarks and registered trademarks belong to the company CEGLA Medizintechnik GmbH & Co. KG.

In some instances, product graphics may vary from actual product models, sizes and/or colors listed below.

Technical and optical changes as well as printing errors reserved.

LATEX FREE

QUANTITY: 1

2680

DO NOT USE IF
DAMAGED

CONSULT
INSTRUCTIONS

KEEP AWAY FROM
CHILDREN UNDER 3 YEARS.
ATTENTION SMALL,
SWALLOWABLE PARTS.

0483

RC-CORNET® PLUS NASAL SET

With the RC-Cornet® PLUS NASAL SET accessory package, RC-Cornet® PLUS can easily be converted for upper respiratory tract therapy.

The delivery package of the RC-Cornet® PLUS NASAL SET includes:

- Elbow connector
(additional to combined therapy see page 22)
- Nozzle

a.


b.

How to convert your RC-Cornet® PLUS:

For conversion into a RC-Cornet® PLUS NASAL please insert the nozzle onto the mouthpiece of RC-Cornet® PLUS.

In order to perform the combined therapy please insert the nozzle onto the elbow connector at first. After that connect this with the mouthpiece of RC-Cornet® PLUS.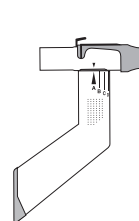

Article no. 2600
PPN 111 241 933 667

Article no. 2683
PPN 111 305 820 282

Article no. 2683
PPN 111 305 820 282

Please read the RC-Cornet® PLUS NASAL instructions from page 14 to 24 carefully prior to using the device for the first time.

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