

DOSING CAPSULES

For clean and easy loading. Dosing Capsules can be prefilled, stored in the Magazine, and inserted into the Filling Chamber before operation. This way, patients have their daily doses contained and portable.

Dosing Capsules are suitable for herbs. Liquids can be used with the additional Liquid Pad.*



Also compatible with the VOLCANO MEDIC 2 in combination with the MEDIC VALVE Dosing Capsule Adapter.

*Cannabinoids dissolved in alcohol are only suitable for the VOLCANO MEDIC 2.

MANUFACTURER



Certificate

No. Q5 063179 0067 Rev. 02

Holder of Certificate: **STORZ & BICKEL GMBH**
In Grubenäcker 5-9
78532 Tuttlingen
GERMANY

Facility(ies): STORZ & BICKEL GMBH
In Grubenäcker 5-9, 78532 Tuttlingen, GERMANY

Certification Mark:



Scope of Certificate: Design and Development, Manufacturing and Distribution of vaporizer systems for release of Cannabinoids out of Cannabis blossoms (Cannabis flos), Cannabis extracts or pure Cannabinoids for medical purpose

Applied Standard(s): EN ISO 13485:2016
Medical devices - Quality management systems -
Requirements for regulatory purposes
(ISO 13485:2016)
DIN EN ISO 13485:2016

The Certification Body of TÜV SÜD Product Service GmbH certifies that the company mentioned above has established and is maintaining a quality management system, which meets the requirements of the listed standard(s). See also notes overleaf.

Report No.: 713185997
Valid from: 2020-06-25
Valid until: 2021-11-20

C.D.M

Christoph Dicks
Head of Certification/Notified Body

Date, 2020-06-25

Page 1 of 1
TÜV SÜD Product Service GmbH • Certification Body • Ridlerstraße 65 • 80339 Munich • Germany



STORZ & BICKEL GmbH · In Grubenäcker 5–9
78532 Tuttlingen · Germany
+49 7461 9697070 · info@vapormed.com
www.vapormed.com



**Certified Manufacturer
ISO 13485 Quality Management**



VAPORMED

Medical Cannabis Inhalation

Volcano
MEDIC 2

Mighty⁺
MEDIC



Certified Quality
ISO 60601 Medical Devices

© by STORZ & BICKEL GmbH · VMAL-40-035 08-2021 · Subject to alterations · All rights reserved

INTENDED USE

The **VOLCANO MEDIC 2** and the **MIGHTY+ MEDIC** are intended for vaporizing and subsequently inhaling cannabinoids. The goal is the application of active ingredients in the human body via the alveoli for the appropriate medical indication. It is suitable for the temporary inhalative application of cannabinoids prescribed by a physician for use at home or in a hospital.

EFFICIENCY

The inhalative application of cannabinoids is by far the most preferred method to ingest cannabinoids. One of the reasons is the rapid onset of effect. Both, the **VOLCANO MEDIC 2** and the **MIGHTY+ MEDIC** ensure an effective inhalation. They vaporize cannabinoids by hot air, without combustion. As a result, no harmful combustibles are produced. Studies have shown that at least 1/3 of the cannabinoids used in the device enter the bloodstream. **VAPORMED** Inhalers have been used in numerous medical studies and have established their reputation as certified medical inhalers for cannabinoids, licensed for Europe, Canada and other countries.*

*More information on www.vapormed.com

MEDIC VALVE BALLOON

The removable Valve Balloon allows a large freedom of movement and an application independent of the basic device. Inhalation out of the Valve Balloon is comfortable due to practically no draw resistance. The **VOLCANO MEDIC 2** also has the option of inhaling directly from an attachable Tube.



VOLCANO MEDIC 2
PPN: 111588550156

MEDICAL AND PORTABLE

MIGHTY+ MEDIC is the first medically approved portable inhaler. It uses a combined air and radiation heater that ensures efficient vaporization from the first breath.

The portable, battery-powered **MIGHTY+ MEDIC** Vaporizer is easy to use. The two high-performance rechargeable batteries ensure a long service life and a short heat-up time. It also has a cooling unit for pleasant inhalation.



MIGHTY+ MEDIC
PPN: 111661680357