KOLOR PLAY[®] KOLOR CLIP[®] by kitett[®]

FR

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GB

INSTRUCTIONS FOR USE

D

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IT

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ES

MANUAL DE INSTRUCCIONES

PT

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NO

BRUKSANVISNING

FI

KÄYTTÖOHJE

AR

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Contents

Instructions for use of KOLOR PLAY® and KOLOR CLIP®

GB

1. Intended Use	p.11
a. Indications	p.11
b. Contraindications and adverse effects	p.11
c. Users	p.11
2. Product description	p.11
a. List of components	p.12
b. Choice of breast shield size	p.12
3. Cleaning / sterilization of the breast pump and accessories	p.13
a. Cleaning	p. 13
b. At-home sterilization	p. 13
4. Use	p.14
a. Preparing the breast pump and its accessories	p.14
b. Operating the breast pump	p.14
c. After each use	p.15
d. Dismantling the breast pump	p.15
5. Breastfeeding	p.15
6. Important recommendations	p.16
7. Troubleshooting	p.16
8. List of accessories / components	p.17
9. Warranty	p.17
10. Technical specifications	p.18
11. Pictograms	p.18

1 Intended Use

a. Indications

In cases where breastfeeding is insufficient or impossible, breast pumps help mothers to ensure their babies always have the benefit of breast milk.

Breast pumps can be used at different stages of lactation. At the beginning, they can help to stimulate the production of milk and later, help to maintain long-term production.

Women may wish to express their milk for a number of reasons, including:

- Medical reasons related to their baby, such as sucking problems, premature birth, neonatal pathologies or cleft lip.
- Medical reasons related to motherhood, such as cracks, engorgement, painful breastfeeding, inverted nipples or medical treatment incompatible with breastfeeding.
- Comfort reasons, for example mothers who wish to store their milk due to temporary absence.
- Reasons related to returning to work, for mothers wishing to continue to feed their baby with breast milk.

b. Contraindications and adverse effects

There is no known contraindication to the use of KOLOR PLAY® manual breast pumps.

Adverse effects: A risk of allergy to the materials used (thermoplastic elastomer, dyes) with the part of the body in contact with the breast shield is possible.

c. Users

The breast pump is intended for use by any breastfeeding woman, whether an adult or a teenager (accompanied by an adult). The minimum requirement is that you read these instructions.

2 Product description

These instructions are designed to inform readers about KOLOR PLAY® and KOLOR CLIP®.

KOLOR PLAY® is a manual breast pump. It is particularly indicated for occasional use. It may be used to supplement an electrical breast pump, in cases of temporary separation or for any other reason making breastfeeding insufficient or impossible.

KOLOR CLIP® includes all the components you need to convert your KOLOR® Expression Kit into a KOLOR PLAY® manual breast pump. You may also use a KOLOR® breast shield already in your possession, for example from having rented a Fisio or Fisio Pro breast pump.

The ergonomics of the KOLOR PLAY® breast pump have been especially designed for mothers, making it easy to handle and simplifying the pumping action. The handle and the bottle are comfortable, designed with the shape of mothers' hands in mind.

All sizes of KOLOR® breast shield fit the breast pump. Mothers may therefore use a breast shield size that suits their breast shape and enjoy optimal comfort for prolonged, sustainable breastfeeding.

KOLOR PLAY® breast pump benefits from «Clean Valve» technology, limiting the risk of bacterial contamination from fingers onto the parts in contact with the breast milk.

KOLOR PLAY® reproduces the **2 physiological phases** of breastfeeding:

- stimulation phase,
- expression phase,

a. List of components

- 1 KOLOR® breast shield
- 2* 'Clean Valve' composed of a transparent valve and a taupe-coloured positioning ring
- 3 Baby bottle
- 4* Breast pump body
- 5* Membrane with base
- 6* Handle
- 7* Bottle stand
- 8 Cap for closing the bottle and keeping the milk after using the breast pump
- 9* Pouch for storing KOLOR PLAY® breast pump accessories (breast shield, baby bottle)

* In the **KOLOR CLIP®** version.

! The «Clean Valve» to be used with the manual breast pump should be taupe-coloured.

Should not be used with a Kitett® electric breast pump.

b. Choice of breast shield size

The KOLOR PLAY® breast pump is for use with the KOLOR® EXPRESSION KIT.

A breast shield is included in the standard KOLOR PLAY® breast pump pack. KOLOR® breast shields are available in 4 inside diameters and 2 «comfort» sizes, that fit perfectly onto all shapes of breast, ensuring the highest level of comfort.

• 4 inside diameters: 4 colour codes

21 **24** **26** **30** mm

• 2 comfort sizes:
S = 76 mm and L = 90 mm

KOLOR® breast shields are guaranteed Phthalate and BPA free.


They can be reused throughout the period of breast pump use.

For optimal hygiene, breast shield is a single unit, to avoid touching the screw thread of the bottle or the internal parts of the bottle and the shield.

Measurements:

Measurements should be taken after breastfeeding or expressing your milk. Alternatively, 'when at rest', add an extra 2mm to your nipple measurement to select the correct inner diameter size.

△ Your nipple should not be too free. You should see space around your nipple in order to let it move easily in the funnel during pumping.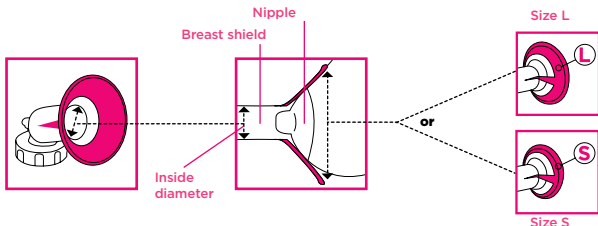

- If the breast shield is too tight: lesions, compressed ducts

- If the breast shield is too wide: less milk will be expressed

A correct comfort size (S or L outer fitting) will improve breast pump pressure (optimal airtightness).

3 Cleaning / sterilization of the breast pump and accessories


• **Before using for the first time and after each use thereafter (and long periods of disuse)**

a. Cleaning

- Rinse out all the elements comprising the breast pump, Kolor teat and feeding bottle in cold water, then wash in hot soapy water.

- Rinse all the elements and assemble the two components comprising the valve: the transparent valve and the positioning ring (making sure that the valve is positioned correctly on the ring).

- Drain on a piece of kitchen paper, then leave to dry in the air.

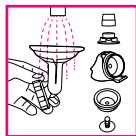! You are advised to wash all parts in contact with breast milk immediately after breast pump use. This makes cleaning easier and prevents the development of bacteria.

• **Before using for the first time and regularly during use:**

b. At-home sterilization

Sterilize all parts in boiling water (10 minutes minimum), in the dishwasher or in a microwave sterilizer. (Recommendation for disinfection with microwave sterilizer : 800w-950w for 4 min)

If you sterilize in boiling water and if you have hard water, you may add a few drops of white vinegar (2 or 3 drops) to avoid deposits forming on the components.

4 Use

a. Preparing the breast pump and its accessories

Illustration 4.a

1-Wash your hands in soapy water before handling breast pump components.

Assemble the handle:


2-Assemble the membrane and the base with the stem. Make sure the base fits correctly against the membrane.

3-Insert the whole assembly into the body of the breast pump and squeeze the rim tightly to ensure a good watertight fit around the membrane.

4/5-Clip the handle onto the body via the notches and horizontal bar (click).

Assemble the accessories:

6-Place the collector bottle directly onto the taupe-coloured "Clean Valve" (the valve should be fixed to a smooth, clean surface). To ensure hygiene, both parts of the valve should be pre-assembled after cleaning: valve and positioning ring. (see chapter 3)

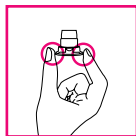If necessary, take care to handle the "Clean Valve" by the rim (taking care not to touch parts that may be contact with the breast milk).

The innovative technology of the «Clean Valve» valve, exclusive to KITETT®, prevents the risk of contamination through contact with fingers in the breast shield.

7-Screw the KITETT Kolor® breast shield onto the collection bottle (ensuring that the taupe-coloured "Clean Valve" is correctly positioned in the bottle)

8-Fit the body/membrane/handle unit onto the breast shield. Make sure everything is interlocked (click).

Place the assembled breast pump on the bottle stand for optimal stability.

b. Operating the breast pump

Illustration 4.b

1-Place the breast shield correctly over your breast, ensuring the milk can flow freely into the bottle.

2-Ensure your nipple is centred inside the duct. The flexible part of the breast shield must also be properly positioned on your breast to ensure complete sealing.

3-Start to pump.

Stimulation phase:

Pump with short, rapid movements to initiate milk let-down.

The handle should be pressed down as far as the grey mark on the body of the breast pump

4 - Expression phase: When the first drops of milk start to flow, start pumping with longer, less frequent movements.

The handle should be pressed down until it touches the bottle.

The speed and length of movements will help you to adjust the strength of suction, according to how you feel.

5 - When milk no longer flows or when the bottle is full, stop pumping.

Place the assembled breast pump on the bottle stand for optimal stability.

c. After each use**Illustration 4.c**

1- Unscrew the breast shield / handle unit from the bottle.

2- Remove the taupe-coloured "Clean Valve". Use the bottle stand to avoid any risk of upsetting the bottle.

3- Place the cap firmly on the bottle. Please refer to the following section regarding the preservation/de-freezing of breast milk. Breast milk can be kept in a refrigerator at 4°C for 48 hours.

4- After use, rinse all parts in contact with milk in cold water. Then wash them in hot and soapy water. Rinse and leave to dry in the open-air on a piece of paper towel.

d. Dismantling the breast pump**Illustration 4.d**

1- Disconnect the handle/body/membrane assembly of the breast shield by twisting slightly on the side

2- Twist the handle upwards to release it from the body of the breast pump.

3- Remove the membrane and its base. Carefully remove the stem from the base, making sure you do not damage the membrane orifice.

4- Dismantle the two parts comprising the valve: the transparent valve and taupe-coloured positioning ring.

5 Breastfeeding

Physiology of baby breastfeeding

KOLOR PLAY® breast pump provides mothers with the natural sensations of breastfeeding by recreating the way in which babies feed.

It reproduces the 2 physiological phases of breastfeeding:

- The stimulation phase to generate milk let-down (short, rapid movements),
- The expression phase to collect milk (longer, less frequent movements).

6 Important recommendations

Use - Functionality

- This device must not be left within the reach of unsupervised children or animals (risks caused by detachable components).
- The device should be installed and started up in accordance with the information supplied in the accompanying documents.
- The device must only be used under the operating conditions indicated by the manufacturer.
- If you notice that the device is not working correctly, contact your supplier.
- This is a fragile medical device, to be handled with care.
- Take care to keep these instructions with the device as they contain the information needed on how to use the equipment correctly.
- The Export version breast shield (with Export screw thread) can only be used with a corresponding bottle (same screw thread).

CAUTION, in all cases:

- WARNING: No modifications permitted.
- WARNING: Only use the accessories supplied and listed in the instructions for use.
- Do not thaw or heat the collected milk in the microwave.
- If signs of degradation or color change appear, please replace the accessories.

7 Troubleshooting

Problem	Solution
Low or non-existent suction on the Expression Kit	Check that the breast shield is fully screwed onto the bottle (screw tightly). Check for presence of taupe-coloured «Clean Valve» specific to manual breast pump. Check the right positioning of the transparent valve on the ring of the 'Clean Valve'. Check the membrane is fixed on its base. Check the membrane and the handle are fixed onto the body of the breast pump. Check the state of the seal between the breast shield and the feeding bottle.

8 List of accessories / components

Part designation	Item code
Fisio Bib - Batch of 2x 100 ml collection receptacles VE	FIBIBN-E
Taupe-coloured KOLOR PLAY® valve («Clean Valve») - Batch of 2	Y1686
Bottle stand - Batch of 2	YSUPBIB
Pouch - Batch of 2	Y1423
Membrane with base and stem - Batch of 2	Y1656
Handle	Y1338
Breast pump body	Y1337
KOLOR MOOV transport bag for breast pump	M1540

		SINGLE Expression Kit	DOUBLE Expression Kit	Téterelle seule
Inner Ø (in mm)	21	K21ES, K21EL, MK21ES	K21ESD, K21ELD, MK21ESD	WB21ES, WB21EL
	24	K24ES, K24EL, MK24EL	K24ESD, K24ELD, MK21ELD	WB24ES, WB24EL
	26	K26ES, K26EL, MK26EL	K26ESD, K26ELD, MK26ELD	WB26ES, WB26EL
	30	K30ES, K30EL, MK30EL	K30ESD, K30ELD, MK30ELD	WB30ES, WB30EL

9 Warranty

This device is guaranteed for 1 year as from date of purchase against all manufacturing defects.

This warranty does not cover the consequences of accidents or any use that does not comply with the manufacturer's instructions.

To be accepted, invoices must include the vendor's stamp and be dated on the day of purchase.

To make a claim under the guarantee, the device must be returned, preferably complete, with the invoice, carriage paid.

10 Technical specifications

GB

Device class	IIa
Pressure	-300 hPa
Weight (with breast shield)	167 gr
Size (with breast shield)	190 x 168 x 90 mm

Ambient temperature (T) For transport and storage		$-25^{\circ}\text{C} < T < +50^{\circ}\text{C}$
Ambient temperature (T) When in operation		$+10^{\circ}\text{C} < T < +40^{\circ}\text{C}$
Relative humidity (H) For transport, storage and operation		$15\% < H < 93\%$

Identification: Each breast pump has a batch ID, which indicates the date of manufacture and the operator code.

Ex: **LOT** 12-2016 46

12: operator code; 2016: year; 46: Week of manufacture.

The EC declaration of conformity can be proved on request

11 Pictograms

Symbols used	
	Compliance with directive 93/42/EEC of 14 June 1993 on medical devices
	Manufacturer
	Refer to the instructions for use
	Device reference
	Batch number
	Store away from direct sunlight
	This symbol indicates the temperature limit which must not be exceeded in the case of operation, transport, and storage
	Moisture limit which must not be exceeded in the case of operation, transport, and storage
	Export screw thread
	BPA free material
	Unique Device Identification
	Medical Device

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