SONIC

PURENEB[®]
EXTRANEB SONIC

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Sonic aerosol nebulizer

Sonic Vernebler

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Sonische aërosolverstuiver

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REF

CE 0459

NL11SN-GB

Nébuliseur avec masque adulte et embout nasal / Nebulizer with adult mask and nasal piece / Vernebler mit Erwachsenenmaske und Nasenendstück / Nebulizador con mascarilla para adultos y boquilla nasal / Nebulizzatore con maschera per adulti e forcilla nasale / Verstuiver met volwassen masker en neusopzetting / Nebulizator z maską dorosły i końcówką do nosa

NL11SNP-GB

Nébuliseur avec masque pédiatrique et embout nasal / Nebulizer with paediatric mask and nasal piece / Vernebler mit Kindermaske und Nasenendstück / Nebulizador con mascarilla para niños y boquilla nasal / Nebulizzatore con maschera per bambini e forcilla nasale / Verstuiver met pediatrisch masker en neusopzetting / Nebulizator z maską dzieci i końcówką do nosa

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EXTRANEB SONIC nebulizer

1. Instructions for use

For a good hygiene, before the first use, sterilize the nebulizer in boiling water (20 minutes). Connect the tube to the "Pressure" outlet of the device and to the inlet (1) of the nebulizer. Connect the big tube to the "Vibrations" outlet of the compressor and to the inlet (7).

Fix preferably the nasal piece, or the mask, to the aerosol outlet (6). Open the upper part of the body of the jet nebulizer unscrewing it by $\frac{1}{4}$ revolution. **Check that the jet-pipe (4) is correctly inserted on the bottom part (3).** Pour the medicine in the tank (2) without overfilling the mark "FULL" (6 ml). Place back the upper part of the body of the nebulizer screwing it by $\frac{1}{4}$ revolution. Put the device ON. A light mist of aerosol must come out of the nebulizer. If it is not the case, check that the jets (3) or (4) are not clogged and that the **upper jet (4) is well positioned down** on the bottom part (3). Do not cover the venture hole (5).

See instructions for use of the device.

This nebulizer cannot be used with a MANOSONIC AMSA compressor.

2. Maintenance and cleaning

After each inhalation session, clean the jets (3) and (4) by using the nebulizer with water. Then, **unplug the tubes coming from the compressor** and wash the nebulizer with running hot water. Do not use any metal wire to clean the jets (3) and (4).

3. Sterilization

Dismantle the body of the nebulizer according to the picture.

Nebulizer, nasal piece, mask and tubes can be sterilized in boiling water (20 min).

Nebulizer itself can be sterilized at 134°C.

4. European Standard NF EN 13544-1 (with AOHBOX+ compressor)

Maximum volume filling : 6 ml

Minimum volume filling : 1 ml

Performances for a 4 ml volume:

Aerosol output (a): 1.25 ml NaF 1%

Aerosol output flow (b): 0.107 ml/mn

MMAD (c): 4.1 μm

(a) Medicine volume inhaled by the patient during one nebulization treatment.

(b) Medicine volume inhaled by the patient during 1 minute of nebulization.

(c) Mass Median Aerodynamic Diameter (size of the aerosols particle).

For hygienic purpose, a brand new EXTRANEB SONIC nebulizer must be given to each patient.

This nebulizer can be used with a SONIC compressor of DTF MEDICAL.

Expected shelf life of accessory: 1 year.

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